


**PRODUCT DATA SHEET**

## CRT Universal 3Ltr Medical Courier Tote


**Model Number:** MCT8S-3L10H-25P

**Description:** Multi-use Medical Courier Tote with 14" x 12" x 11" external dimensions and 3 Liter payload volume. Validated to keep payload between +15°C and +25°C for 10 hours in Summer per custom cycle with 15 second opening every 15 minutes.

**Exploded View**

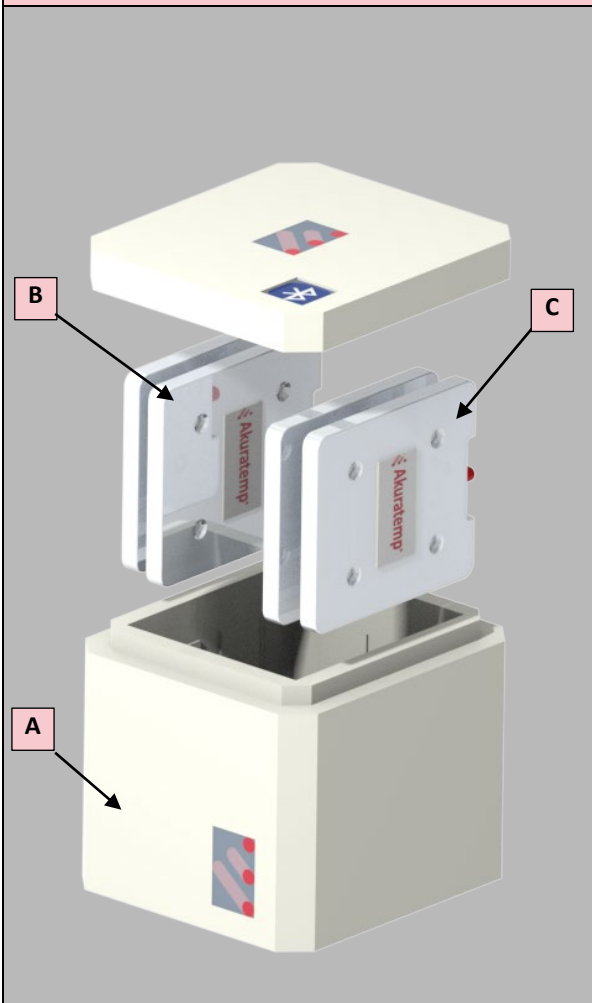

**Temperature Graph per Custom Summer Profile**

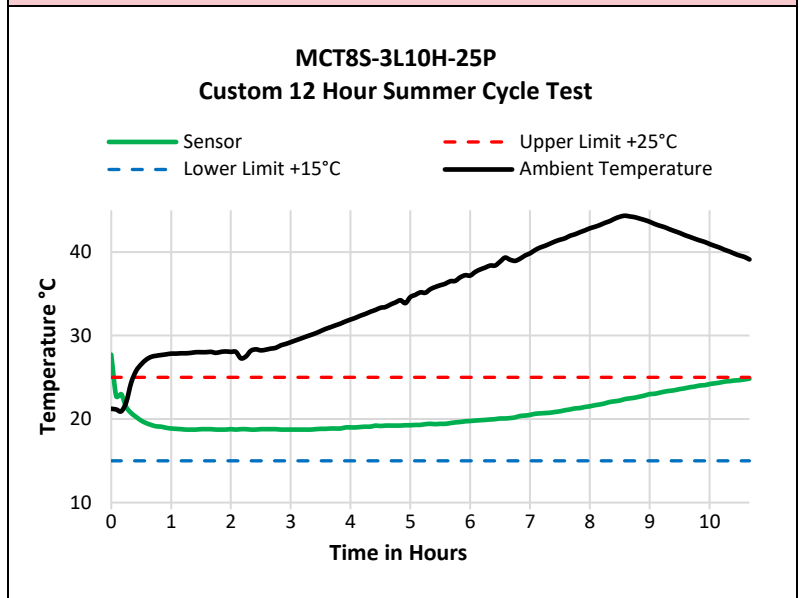

**Holdover Time per Custom Summer Profile**

Test	Open	Payload Temperature	Holdover Time
Summer Cycle	15 sec per 15 min	Below +25°C	10 hours
Winter Cycle	-	-	-

**Product Specifications**

<b>External Dimensions</b>	14" x 12" x 11"		<b>Payload Dimensions</b>	7" x 7" x 3"	
<b>Payload Volume</b>	3 Liters	<b>Actual Weight</b>	10 lbs.	<b>Dimensional Weight</b>	16 lbs.
<b>Pallet Quantity</b>	84	<b>Akuratemp Compatible</b>	Yes	<b>Warranty</b>	12 Months

#	Bill of Materials		Weight	Qty	Conditioning Instructions
A	ML12121111	PUF Shipper	1.75 lbs.	1	Store at room temperature of +20°C to +25°C for at least 24 hours prior to use.
B	OM18P-8X7-500	PCM CRT Pack	1.45 lbs.	2	Condition at room temperature of +20°C to +25°C for 24 hours prior to use.
C	HS01P-8X7-600	PCM Cold Pack	1.65 lbs.	2	Condition in a refrigerator operating at +5°C for 24 hours prior to use.
D	Insulated Tote Bag	MCTS8S-TOTE	1.60 lbs.	1	Store at room temperature of +20°C to +25°C for at least 24 hours prior to use.